DE:

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EN:

NAWA® MEDICAL ELEKTROLYT-SALBE S is a wound healing cream for external use for the healing of injuries to the skin.

- Promotes wound healing • Reduces swelling • Alleviates pain • Reduces scar formation • Treatment of scar tissue • Infection prevention • Suitable also for children • for wound edge protection • for skin protection • creates an optimal physical environment • reduction of bruising and edema • supported the new tissue formation • doping-free • suitable for sensitive skin • nourishes • soothes • reduces redness of the skin • reduces the feeling of tension • soothes irritations • relieves itching, burning • cooling

Indications: Injuries of the skin (e.g. cuts, abrasions, lacerations and contusions), 1st and 2nd degree burns (also for sunburns), irritation or inflammation of the skin at the anus (e.g. anal fissures, hemorrhoids stadium 1), for the treatment of scar tissue and blunt traumas. Also, for the care of exposed or dry, cracked and aged skin; for the care of the wound edge and for the protection of the skin.

Characteristics: NAWA® MEDICAL ELEKTROLYT-SALBE S creates an optimal physical environment for the healing of injuries to the skin. Thus hematoma and edema were degraded, healing accelerated, swelling reduced and thereby pain relieved. The formation of new tissue is supported, the healing after surgeries or injuries (wounds, burns, sunburns) enhanced and scar formation reduced.

Directions for use: NAWA® MEDICAL ELEKTROLYT-SALBE S is suitable for children over 2 years of age, adolescents up to 18 years of age and all patients over 18 years of age. Wash hands before and after treating a wound. If necessary clean the wound carefully (e.g. with NAWA® Wundspray). Before each application to the anus, it should be cleaned and dry. Apply NAWA® MEDICAL ELEKTROLYT-SALBE S generously onto wound area and over the wound margin. Repeat several times a day for the first three days after injury. The duration of use is based upon type and severity of the injury. NAWA® MEDICAL ELEKTROLYT-SALBE S may be used in combination with a customary dressing. By covering injured skin with a bandage the wound is protected additionally from external influences which could disturb the healing process. NAWA® MEDICAL ELEKTROLYT-SALBE S contains no skin irritating substances and can be used on the mouth and on mucous membranes (e.g. nose).

Contraindication and undesirable side effects: Do not use with hypersensitivity to any of the ingredients.

Warning instructions: For external use. Do not swallow. Look after hygienic extraction of the salve. Close tube after use to avoid contamination. Dispose the tube immediately after contact of the tube opening with the wound or in the case of otherwise contamination. Do not use damaged packages. Do not use if packaging is damaged. Keep out of the reach of children. A doctor should be consulted with burns second b or third degrees. Application for pregnant or breastfeeding women, babies and toddlers/ young children should only be carried out with mandatory indication after consultation of a doctor. Protect from sunlight. Do not use after expiration date. Consider temperature limit. Keep in a dry place. Consider instructions for use. All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user is established.

Shelflife: 3 years after manufacturing

package sizes: Sachet 5 g, Tuben: 20 g, 30 g, 75 g/ 75 ml, 100 g/ 100 ml

Time limit to use: tube: Once opened NAWA® MEDICAL ELEKTROLYT-SALBE S should be used within 3 months. / **Sachet 5g:** The product NAWA® MEDICAL ELEKTROLYT-SALBE S Sachet is intended for single use only. Once opened NAWA® MEDICAL ELEKTROLYT-SALBE S Sachet should be used immediately to avoid contamination.


Batch code/  **and Expiry date/**  : printed onto bottom of folding box and stamped onto tube fold/sachet edge

Disposal: The medical device and any used dressing materials can be disposed in the residual waste; separate disposal is not necessary.

Inhaltsstoffe/Ingredients (INCI):

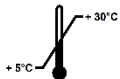Aqua purificata, Octyldodecanol, Hydrogenated Palm Glycerides, PEG-200 Hydrogenated Glyceryl Palmate, Cetyl Alcohol, Methylparaben (and) Propylparaben (and) Sorbic Acid (and) Dehydroacetic Acid, Trace elements (zinc, iron)

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NAWA® HEILMITTEL GmbH