## DE

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# **Activity First Aid Spray**

**EN**

**NAWA® Activity First Aid Spray** is a spray for moisturizing, cleansing and healing of acute wounds.

**Characteristics:**

**NAWA® Activity First Aid Spray** is used for moisturizing and cleansing of acute wounds and supports wound healing (prevention of infections due to acid pH).

moisturizing of acute wounds	Cleansing the affected area	Reduction of swelling
Reduces bleeding	Supports wound healing	Prevents infections
Suitable for children and sensitive skin	Prevents scarring*	For the pain-free wound cleansing

\*if the wound area is covered with a bandage and kept moist with **NAWA® Activity First Aid Spray** until the wound is closed

**Indications:**

For acute and initial care for minor, open injuries (e.g. cuts, abrasions, lacerations) and 1st and 2nd degree burns.

**Directions for use:** **NAWA® Activity First Aid Spray** is suitable for children over 2 years of age, adolescents up to 18 years of age and all patients over 18 years of age. Wash hands before and after use. Cleanse and moisturize wound with **NAWA® Activity First Aid Spray**. Apply directly out of the bottle into the wound and und if necessary, repeat several times. **NAWA® Activity First Aid Spray** can be used in combination with customary wound dressings. If possible, cover the wound area with dressing and keep wet with **NAWA® Activity First Aid Spray** until the wound is closed. Make sure that the dressing does not stick to the wound. Subsequent treatment from day 3 recommended with **NAWA® MEDICAL Elektrolyt-Salbe S**. For a more cooling effect store **NAWA® Activity First Aid Spray** in the refrigerator before application. Can be used in the mouth and mucous membrane areas (e.g. nose). The sprayer with overhead function allows spraying from any position. Doping-free and suitable also for children. The duration of use is based upon type and severity of the injury.

**Contraindication and undesirable side effects:**

Do not use with hypersensitivity to any of the ingredients.

**Tissue tolerance and biocompatibility:**

**NAWA® Activity First Aid Spray** does not contain skin-irritating ingredients. It is non-cytotoxic, non-irritating and non-sensitizing.

**Warning instructions:**

For external use only. Do not swallow. Look after hygienic extraction. Close bottle after use to avoid contamination. Dispose the bottle immediately after contact of the bottle with the wound or in the case of otherwise contamination. Keep out of the reach of children. A doctor should be consulted with burns second b or third degrees. Application for pregnant or breastfeeding women, babies and toddlers/ young children should only be carried out with mandatory indication after consultation of a doctor. Consider temperature. Consider instructions for use. Do not use after expiration date. Do not use damaged packages. Protect from sunlight. Store in a dry area.

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user is established.

**Expiration date:** 3 years after manufacturing      **Time limit to use:** 6 months after opening

**Packaging sizes:** 50 ml, 200 ml plastic bottle

**Batch code/**  : printed onto bottom of bottle

**Expiry date /**  : printed onto bottom of bottle

**Ingredients (INCI):** Aqua purificata, Trace Elements (Zinc, Iron)

**Disposal:** The ACTIVITY device and any used dressing materials can be disposed in the residual waste; separate disposal is not necessary.

 **NAWA HEILMITTEL GmbH**  
Ostendstrasse 100  
D-90482 Nürnberg  
Telefon: +49 (0) 911-544 41-0  
Telefax: +49 (0) 911-544 41-11  
E-Mail: info@nawa.com

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