DE

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Inhaltsstoffe (INCI):

Aqua purificata, Hydroxypropyl Starch Phosphate, Cocamidopropyl Betaine, Benzyl Alcohol (and) Methylchloroisothiazolinone (and) Methylisothiazolinone, Polyaminopropyl Biguanide, Trace Elements (Zinc, Iron)

EN

Hydrogel for covering and moisturizing of acute and chronic wounds

- Characteristics:**
- For covering and moisturizing of dry persistent coated wounds
 - For repeated use

- Indications:**
- Covering of acute and chronic wounds
 - Moisturizing persistent coated dry and low exudating wounds

Directions for use:

Rinse/Irrigate and cleanse wound first with NAWAlution to remove easy solvable contaminations. Thereafter cover the wound surface 3-5 mm thick with NAWA Hydrogel or fill existing wound cavities.

Also wound filler, e.g. alginate or hydro fiber, can be moistened with NAWA Hydrogel and inserted into the wound. NAWA Hydrogel can be used in combination with customary wound dressings, e.g. PU-foams and compresses. NAWA Hydrogel can be left on the wound till next dressing change. The treatment should be repeated till all contaminations can be removed easily and the wound seems visually clean.

Contraindication:

Do not use with known allergy or suspicion of allergic reactions to one of the ingredients. Do not use in combination with anionic tensids. Do not use in combination with silver-containing dressings. Avoid contact with cartilage-tissue. Do not cover wound edges with NAWA Hydrogel because of risk of maceration.

In the case of big, heavily wetting or infected wounds, contact a doctor, because in this case a special treatment is necessary. Contact a doctor for burns of degree 2b or higher.

Application for pregnant or breastfeeding women, babies and toddlers/ young children should only be carried out with mandatory indication after consultation of a doctor.

Side effects: Not known.

Tissue tolerance and biocompatibility:

NAWA Hydrogel does not contain skin-irritating ingredients. It is non-irritating and non-sensitizing.

General safety instructions:

Consider temperature limit. Do not use damaged packages. Only use faultless and undamaged packages. Protect from sunlight. Keep in a dry place. Consider instructions for use. Do not use after expiration date. For external use only. Do not swallow. Mind hygienic use. Close after use to avoid contamination. Dispose the tube immediately after contact of the tube opening with the wound or in the case of otherwise contamination. Keep out of the reach of children.

Shelf life: 3 years after manufacturing

Time limit to use: 4 weeks after opening

Packaging sizes:

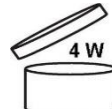
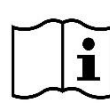
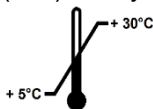
e 10 x 5 ml plastic tubes	REF: 1235849510
e 40 ml plastic tube	REF: 1235849040
e 50 ml plastic tube	

LOT-Number/:  printed onto folding box and stamped onto tube fold

Expiration date/:  printed onto folding box and stamped onto tube fold

Ingredients (INCI):

Aqua purificata, Hydroxypropyl Starch Phosphate, Cocamidopropyl Betaine, Benzyl Alcohol (and) Methylchloroisothiazolinone (and) Methylisothiazolinone, Polyaminopropyl Biguanide, Trace Elements (Zinc, Iron)

CE 0197**NAWA Heilmittel GmbH**

Ostendstrasse 100 • D-90482 Nürnberg

Telefon: +49 (0) 911-544 41-0

Telefax: +49 (0) 911-544 41-11

E-Mail: info@nawa.comwww.nawa-medical.comMade in Germany
REV 032020